•ANTROSOND

Dr. C. Antropoli

sapi med
SURGICAL
LINE

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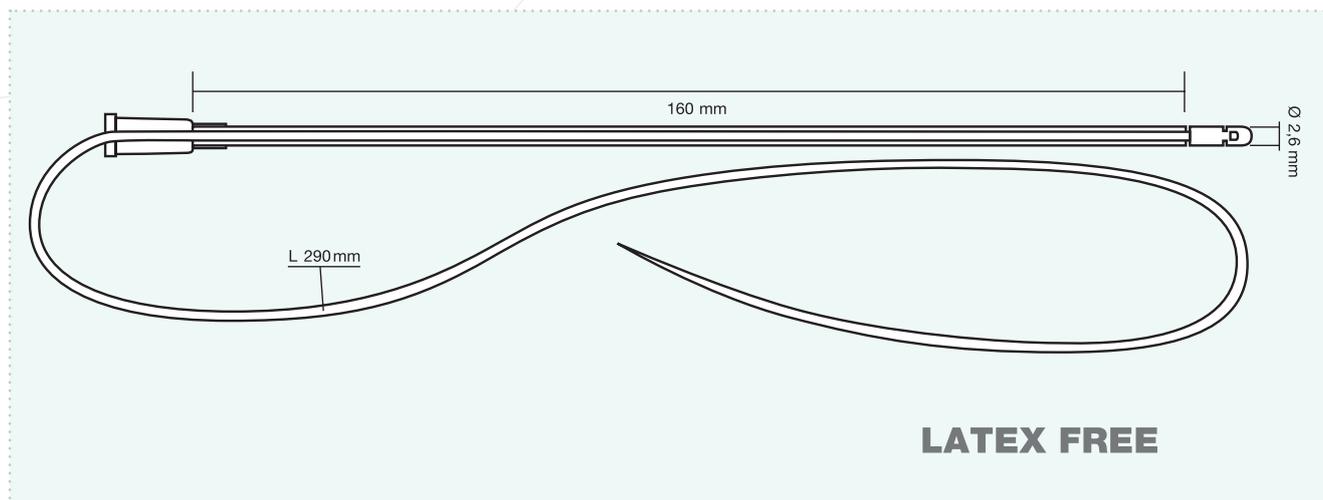
Antrosond has been designed to assist in the perfection of a widely used technique employed in the treatment of fistulas, and particularly perianal fistulas.

The technique itself is extensively supported by evidence of positive outcomes.

The Antrosond device is sterile and disposable, designed to explore fistulas and simultaneously introduce silicon drains for slow traction of the sphincter. This combined action reduces intervention time, allows drainage to be precisely positioned, and eliminates the risk of thread breakage due to the pulling tension unavoidable with traditional systems.

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Antrosond consists of a flexible transparent tube 160mm long, with an external diameter of 2,6mm, combined with a 290mm non-allergic biomedical silicon thread (Mono Drain) inside. One end of the tube terminates in a universal female luer lock, to which a syringe may be connected to inject normal saline solution if required. One end of the tie has a rigid oval cap that engages with the tube to make a single integral device.

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INSTRUCTIONS FOR USE

Antrosond is used like a normal probe. When the fistula has been identified, insert the hard oval end of the Antrosond into the fissure. Proceed slowly along the entire tract, and hold the end of the silicone thread near the entrance hole with one hand, and slide the tube out backwards with the other. The tie is now correctly inserted and positioned for optimal drainage. After knotting the two ends you can start to place the muscle bands under traction.

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Quantity per box: 10 pcs.

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Sterilised by: ETO